

# Instructions for Cleaning & Sterilizing

## doctoreyes cheek retractors



doctoreyes cheek retractors are medical devices of class 1 with a very smooth surface. material: Transparent plastic Polypropylene (PP) or Polycarbonate (PC).

Whilst the usual protocols for cleaning all dental instruments apply, if these instructions are followed and damage avoided, you should obtain long lasting exceptional cheek retractor performance.

### sterilization / disinfection

It is not necessary to sterilize doctoreyes cheek retractors in an autoclave. Following thorough cleaning, wiping with disinfectants such as ID 212 forte, BIB forte or Lysetol is satisfactory provided the manufacturers' instructions are followed precisely.

Strict hygiene laws are in place in Germany where these cheek retractors are manufactured and German dentists use the methods described above which is very effective on the plain smooth surface.

**DO NOT USE AGGRESSIVE OR ACIDIC DISINFECTANTS AS USED FOR OTHER INSTRUMENTS**

### Care instructions

- doctoreyes cheek retractors can be placed in, thermoisinfectant up to 90°C or in autoclaves up to 130°C

- If necessary, remove deposits with a mild detergent or soft cloth. Do not use hard brushes, rough and scratching sponges, or the like.

- Place the mirrors in a disinfectant solution immediately after use and clean at once to avoid sticky deposits encrusting on the surface.

- Wait until the autoclave has fully completed its cycle

- Dry the cheek retractors with a lint-free cloth.

### IMPORTANT – TO AVOID CRUSTS AND RESIDUES ON CHEEK RETRACTORS

doctoreyes cheek retractors must be cleaned with fresh distilled water. If tap water is used, the mirrors must be dried thoroughly afterwards.

If a thermoisinfectant is used, ensure that it is descaled regularly. Avoid short programmes with rapid temperature changes.

## doctoreyes

Abteistrasse 28

88416 Ochsenhausen

phone: +49 7352 9392-12

Email: [info@doctoreyes.de](mailto:info@doctoreyes.de)

Homepage: [www.doctoreyes.de](http://www.doctoreyes.de)